EN
PRODUCT NAME
Particulate Protective Respirator
MODEL NO.
MEGA-KN002
EXECUTIVE STANDARD
EN149:2001+A1 2009
PRODUCT CLASSIFICATION
FFP2 NR
CERTIFICATE ISSUING BODY
The conformity assessment of the regulation (EU) 2016/425 conducted by: SGS Fimko Oy, Takomotie 8, FI 00380 Helsinki, Finland. Notified Body No. 0598
PRODUCT FUNCTION AND COMPLIANCE STATEMENT
This product is designed to provide adequate sealing on the face of the wearer to against both solid and liquid aerosols from the ambient atmosphere, and protect respiratory system, it has passed EN149:2001+A1:2009 to show conformity to the EHSR of Regulation (EU) 2016/425.
PRODUCT USAGE RESTRICTIONS
Cover the mouth, nose and jaw of the user, wear it in the environment of daily life and protect respiratory system from solid and liquid aerosols.
HOW TO WEAR
1. Unpack the mask and check whether it is in good condition.
2. The marking should be on the outside and the letters of mar king should be facing upward when placing on your face. Avoid the hand contacting the inner side of the mask. Place the mask on the face pull the ear band behind, hang it over the ear and adjust to be comfortable.
3. Press the nose clip gently to make it fit the nose, and then tight the nose clip while pulling the mask down to the jaw.
4. Cover the mask with both hands as much as possible and conduct positive and negative pressure tests to make sure that the mask fits you well.
MANUFACTURER
MEGASOFT (CHINA) CO., LTD.
MANUFACTURER ADDRESS
NO.1, XINQIANG ROAD, MACHINERY PRINTING BASE, GAOXIN DISTRICT, HONGSHAN TOWN, SHISHI CITY, FUJIAN PROVINCE, CHINA.

CONTRAINDICATION, PRECAUTIONS AND WARNINGS.
1. This product is a disposable product which is strictly prohibited to be reused. After each shift of use, it shall be handled according to the requirements of the hospital and the environment protection department.
2. Please check whether the package is in good condition, if damaged, do not use it. The external labelling, production date and validity period shall be confirmed and used within the validity period.
3. Use it with caution in case of non-woven fabric allergy.
4. Replace in case of obvious respiratory resistance, damage or pollution (in case of bloodstain, droplets etc.)
TRANSPORTATION
Prevent from pressure, direct sunlight and rain and snow.
STORAGE
In dry, ventilated environment without corrosive gas and keep away from fire sources and flammables.
WARNINGS
Warning! Face masks are not intended for long term use.
Warning! It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.
Warning! Fit of particle filtering half mask must be checked prior to use.
Warning! Air quality might affect the service life of product.

Respirator EN149:2001 FFP2 INSTRUCTIONS
IMPORTANT: Facial protectors fall into three different protection classes for solid particles: FFP1, FFP2 and FFP3. Check which class your respirator falls under before you continue reading these instructions. You can find the information on both the respirator itself and on its packaging.

USE
FFP1: Respirators which fall under this class of protection protect against solid particles which are toxic. The level of impurities may be no more than a maximum of four times the threshold limit value (TLV)*. FFP2: Respirators which fall under this class of protection protect against solid particles which are toxic. The level of impurities may be no more than a maximum of 10 times the threshold limit value (TLV)*. FFP3: Respirators which fall under this class of protection protect against solid particles which are toxic. The level of impurities may be no more than a maximum of 20 times the threshold limit value (TLV)*. *Threshold limit values are published under the term HTP (Haitalliseksi tunnetut pitoisuudet) by the Finnish Ministry of Social Affairs and Health.

1. Failure to comply with the instructions and limitations of this product can reduce the effectiveness of the respirator and cause illness or risk of death.
2. For the sake of your health, it is important that you choose the correct type of respirator. Contact your occupational hygienist or an occupational health and safety professional before using the respirator to ensure that the product is suitable for use in the desired application.
3. This respirator does not provide oxygen and should not be used in atmospheres containing less than 19.5% oxygen.
4. Do not use the respirator in spaces where the levels of impurities in the air pose an immediate threat to your health or your life.
5. Exit the working area immediately and seek fresh air if you have problems breathing, if you feel faint, or if you feel otherwise unwell.
6. Never modify your respirator or use it in a manner which does not comply with instructions.
7. This product is disposable. Dispose of the respirator after use.
8. Replace your respirator immediately if you have trouble breathing or if the respirator is damaged, faulty or does not sit properly on your face.
9. Store unused respirators in their packaging in a clean and dry place with no air impurities.
10. The respirator may not be used in explosive environments due to the aluminium nose clip.

Limitations:
a. Do not use in atmospheres containing less than 19.5% oxygen.
b. Do not use in environments which pose an immediate threat to your health or life.
c. The inappropriate use or storage of the product may lead to injury or death.
d. Never replace, modify, add or remove components which belong to the product and are determined for it by the manufacturer.
e. Be sure to comply with the instructions for use and storage provided by the manufacturer when you use and store the respirator.

Before use, check that:
a. The straps are firmly attached to the respirator.
b. The nose clip and padding are firmly attached to the respirator.
c. The expiration date is marked on the packaging.

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