EN

PRODUCT NAME
Particulate Protective Respirator

MODEL NO.

KY-8604

EXECUTIVE STANDARD

EN149:2001+A1:2009

PRODUCT CLASSIFICATION

FFP1 NR

CERTIFICATE ISSUING BODY

The conformity assessment of the regulation (EU)2016/425 conducted by: CCQS Certification Services Limited, Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin 15 D15 AKK1, Dublin, Ireland. Notified Body No. 2834

PRODUCT FUNCTION AND COMPLIANCE S TATEMENT

This product is designed to provide adequate sealing on the face of the wearer to against both solid and liquid aerosols from the ambient atmosphere, and protect respiratory system, it has passed EN149:2001+A1:2009 to show conformity to the EHSR of Regulation (EU) 2016/425.

PRODUCT USAGE RESTRICTIONS

Cover the mouth, nose and jaw of the user, wear it in the environment of daily life and protect respiratory system from solid and liquid aerosols.

HOW TO WEAR

- Hold the respirator so that the metal nose clip points upwards. Let the straps hang freely.
- Place the respirator underneath your chin so that it covers your mouth and nose.
- Pull the lower strap over your head and position it below your ears. Pull the upper strap over your head and position it above your ears to ensure a proper fit.
- Adjust the metal nose clip so that it sits tightly around your nose.
- Check the fit of the respirator by pressing it against your face with both hands and inhaling and exhaling deeply. If you feel the flow of air around your nose, adjust the nose clip so that it is tighter around your nose. If air leaks from the edges of the respirator, adjust the straps for a better fit. Re-check the fit of your respirator and repeat the aforementioned procedures until the respirator sits firmly on your face.

MANUFACTURER

Jinhua Kangyi Labor Protection Appliance Co., Ltd.

MANUFACTURER ADDRESS

Shankoufeng Village, Chisong Town, Jingdong District, Jinhua City, Zhejiang Province, China

CONTRAINDICATION, PRECAUTIONS AND WARNINGS.

- This product is a disposable product which is strictly prohibited to be reused. After each shift of use, it shall be handled according to the requirements of the hospital and the environment protection department.
- P lease check whether the package is in good condition, if damaged, do not use it. The external labelling, production date and validity period shall be confirmed and used within the validity period.
- Use it with caution in case of non-woven fabric allergy.
- Replace in case of obvious respiratory resistance, damage or pollution (in case of bloodstain, droplets etc.)

TRANSPORTATION

Prevent from pressure, direct sunlight and rain and snow.

STORAGE

In dry, ventilated environment without corrosive gas and keep away from fire sources and flammables.

WARNINGS

- Warning! Face masks are not intended for long term use.
Warning! It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.
Warning! Fit of particle filtering half mask must be checked prior to use.
Warning! Air quality might affect the service life of product.

Respirator EN149:2001 INSTRUCTIONS

IMPORTANT: Facial protection falls into three different protection classes for solid particles: FFP1, FFP2 and FFP3. Check which class your respirator falls under before you continue reading these instructions. You can find the information on both the respirator itself and on its packaging.

USE

- FFP1: Respirators which fall under this class of protection protect against solid particles which are atoxic. The level of impurities may be no more than a maximum of four times the threshold limit value (TLV)*.
FFP2: Respirators which fall under this class of protection protect against solid particles which are atoxic. The level of impurities may be no more than a maximum of 10 times the threshold limit value (TLV)*.
FFP3: Respirators which fall under this class of protection protect against solid particles which are atoxic. The level of impurities may be no more than a maximum of 20 times the threshold limit value (TLV)*.
*Threshold limit values are published under the term HTP (Haitalliseksi tunnetut pitoisuudet) by the Finnish Ministry of Social Affairs and Health.

- Failure to comply with the instructions and limitations of this product can reduce the effectiveness of the respirator and cause illness or risk of death.
- For the sake of your health, it is important that you choose the correct type of respirator. Contact your occupational hygienist or an occupational health and safety professional before using the respirator to ensure that the product is suitable for use in the desired application.
- This respirator does not provide oxygen and should not be used in atmospheres containing less than 19.5% oxygen.
- Do not use the respirator in spaces where the levels of impurities in the air pose an immediate threat to your health or your life.
- Exit the working area immediately and seek fresh air if you have problems breathing, if you feel faint, or if you feel otherwise unwell.
- Never modify your respirator or use it in a manner which does not comply with instructions.
- This product is disposable. Dispose of the respirator after use.
- Replace your respirator immediately if you have trouble breathing or if the respirator is damaged, faulty or does not sit properly on your face.
- Store unused respirators in their packaging in a clean and dry place with no air impurities.
- The respirator may not be used in explosive environments due to the aluminium nose clip.

Limitations:

- Do not use in atmospheres containing less than 19.5% oxygen.
- Do not use in environments which pose an immediate threat to your health or life.
- The inappropriate use or storage of the product may lead to injury or death.
- Never replace, modify, add or remove components which belong to the product and are determined for it by the manufacturer.
- Be sure to comply with the instructions for use and storage provided by the manufacturer when you use and store the respirator.

Before use, check that:

- The straps are firmly attached to the respirator.
- The nose clip and padding are firmly attached to the respirator.
- The expiration date is marked on the packaging.

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VALMISTAJA

Jinhua Kangyi Labor Protection Appliance Co., Ltd.

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Shankoufeng Village, Chisong Town, Jingdong District, Jinhua City, Zhejiang Provin-
ce, China

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EU DECLARATION OF CONFORMITY

Applicant: Jinhua Kangyi labor Protection Appliance Co.,Ltd.

Address:Shankoufeng Village,Chisong Town,Jingdong District,Jinhua City ,
Zhejiang Province,China

Declares that the new PPE described hereafter

Dust Mask FFP1 without valve (KY-8604)

is in conformity with the provisions of EN149:2001+A1:2009 Respiratory
protective devices - Filtering half masks to protect against particles -
Requirements,testing,marking

is identical to the PPE which is the subject of EU certification of conformity C
E-PC-200311-036-01-9B issued by CCQS Certification Services Limited,

Block 1 Blanchardstown Corporate Park,Ballycoolin Road, Blanchardstown,
Dublin 15,D15 AKK1,Ireland

The importer shall commit to strictly abide by the agreement and not use the
face masks it purchases for medical purposes and to warn any third party
against using the face masks for medical purposes. The exporter or the
producer is not liable for any losses caused by the inappropriate use,
maintenance or keeping of the face masks by the importer or any third party.

Done at Place : Jinhua ,Zhejiang ,China

Date 2020.12.16